sanity®

Inhalator DLA DZIECI

Model: AP 2516

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CONTENT

PURPOSE.....	14
INTRODUCTION.....	15
PRODUCT IDENTIFICATION	16
SYMBOLS USED IN THE MANUAL AND ON THE BOX.....	16
Product Cautions	17
Operating Cautions	17
IMPORTANT SAFEGUARDS	17
Storage Cautions.....	17
Cleaning Cautions	17
OPERATING YOUR COMPRESSOR NEBULIZER	18
CLEANING.....	19
Rinsing (after each treatment).....	19
Disinfections.....	19
Cleaning the compressor	19
Filter change	19
REPLACEMENT PARTS & OPTIONAL ACCESSORIES	20
SPECIFICATIONS.....	20
CONTACT	20
INFORMATION CLAUSE.....	21
THERAPY PROGRESS.....	22
WARRANTY	23

PURPOSE

English version

Thank you for purchasing our compressor nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your compressor Nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

INTRODUCTION

About Sanity®

Sanity® has been present on the Polish market for 20 years as a brand of quality and functional medical products, aiding the return to health, daily health prevention and care for the whole family. Sanity's® product portfolio includes a wide range of medical devices and accessories used to perform basic diagnostic tests, ENT and rehabilitation procedures, as well as treatments supporting daily body care and care for the health and hygiene of adults and children.

Sanity's® products are characterised by their high quality and practical, user-friendly solutions.

Our products comply with the most stringent safety standards, have successfully passed clinical tests and received the corresponding certificates confirming compliance with the EU standards.

Our values

Love is the basis of our humanity. It drives us in life and causes that we become better.

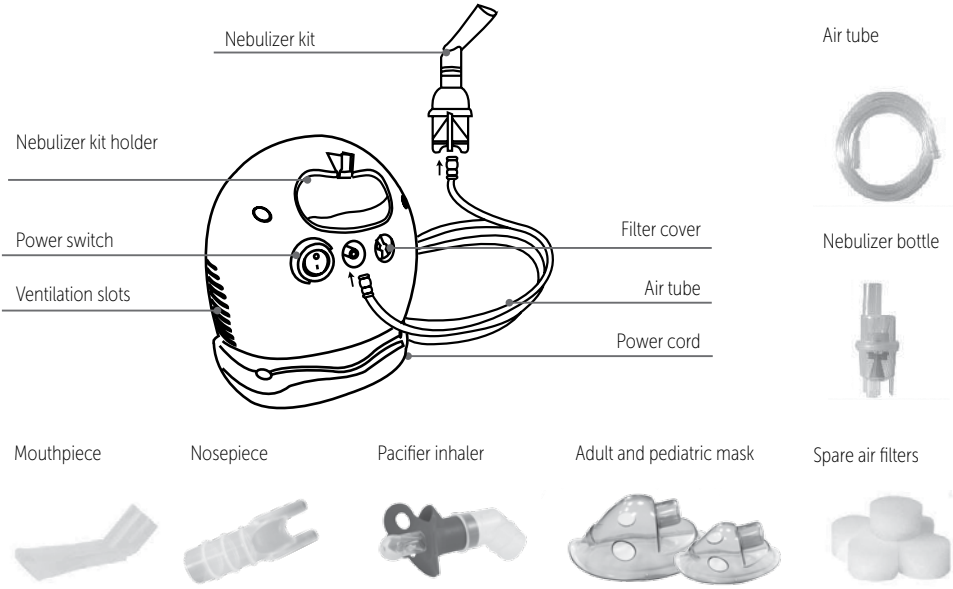
Care is the effect of our feelings and sensitivity. Caring for our close ones guides our actions and changes the way we look at everyday objects.

Help is a sign of love and caring. It includes all, even smallest tasks that we perform for the other person, caring about them in every situation.

PRODUCT IDENTIFICATION

SYMBOLS USED IN THE MANUAL AND ON THE BOX

 Symbol for "TYPE BF APPLIED PARTS"	 Protection against electric shock – protection class II
 Switch OFF	 Keep dry
 Switch ON	 Carefully read the instructions before using this device
 Alternating current	 Important safety instructions
 Symbol for "MANUFACTURER"	 Symbol for „SERIAL NUMBER“
CE 0197 Symbol for "COMPLIES WITH MDD 93/42/EEC REQUIREMENTS"	
 Symbol for "ENVIRONMENT PROTECTION – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice".	

IMPORTANT SAFEGUARDS

Note: Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product.

Caution! Failure to read and observe all precautions could result in personal injury or equipment damage.

Product Cautions

1. To avoid electrical shock:
 - Keep the unit away from water.
 - Do not immerse the power cord or the unit in liquid.
 - Do not use while bathing.
 - Do not reach for a unit that has fallen into water – immediately unplug the unit..
2. Never operate the unit if it has any damaged parts (including power cord), if it has been dropped or submersed in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
5. Do not use other attachments unless recommended by manufacturer.

Operating Cautions

1. Connect this product to an appropriate voltage outlet for your model.
2. Do not run this product unattended.
3. Never operate if this unit has a damaged cord or plug, if it has been dropped, or in any way, into water, if it does not work properly. Return it to a service center for repair.
4. If any abnormality occurs, discontinue use immediately until the unit has been examined and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit or place it where the air openings may be obstructed.

Storage Cautions

1. Do not store the unit under direct sunlight, high temperature or humidity.
2. Keep the unit out of reach of small children.
3. Always keep the unit unplugged while not in use.

Cleaning Cautions

1. Do not immerse the unit in water. It may damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guidebook.

OPERATING YOUR COMPRESSOR NEBULIZER

Caution!

The product must be used in accordance with the recommendations described in the operator's manual.

The user may not implement any changes to the product.

Dear Customer – like any electrical device, our product may disturb the operation of other electrical devices, such as cell phones, wireless telephones, routers – safe distance from other electrical equipment with regard to device operation is approximately 1 meter.

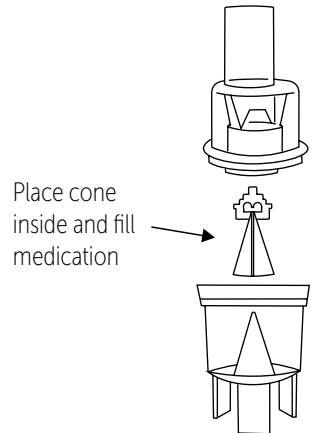
Caution!

Prior to initial operation, the nebulizer should be thoroughly cleaned.

1. Place your compressor nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.

Important: Prior to initial operation, the nebulizer should be thoroughly cleaned referring to the "Cleaning Procedures" in the manual.

2. Gently twist the top part of the nebulizer bottle counter-clockwise to disassemble the nebulizer bottle.
3. Fill the bottom section of the nebulizer bottle with the medication prescribed by your physician. Be sure that the cone is put inside the bottom section and an upper part is yellow sleeve with the tabs pointing upwards.
4. Gently twist the top part clockwise to re-assemble the nebulizer. Be sure the two sections fit well.
5. Attach one end of air tube to the base of the nebulizer bottle.
6. Attach the other end of air tube to the air tube connector located on the front of the compressor.
7. Attach mouth piece or mask per your choice to the top section of nebulizer.
8. Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at "OFF" status.
9. Press power switch to begin your prescribed treatment.
10. After the end of inhalation, turn the machine off and turn it from the mains.

Place cone inside and fill medication

Important: The compressor motor has a thermal protector which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- a. Set the switch in the OFF position.
 - b. Unplug the unit from the electrical outlet.
 - c. Wait 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed.
11. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

CLEANING

It is recommended that the nebulizer, mouthpiece and mask are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Rinsing (after each treatment)

1. Disconnect the air tube, nebulizer, mouthpiece and mask.
2. Gently twist the nebulizer to open it.
3. Rinse the nebulizer, mouthpiece and mask with water.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

Disinfections

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece and mask.
2. Complete Rinsing Steps 1-3.
3. Wash nebulizer, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.
4. Submerge these parts in the vinegar and water solution for thirty minutes.
5. Complete Rinsing Steps 3-5.

Cleaning the compressor

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads, which may damage the finish.

Filter change

1. Do not use cotton or any other materials. Do not wash or clean the filter. Use only 's filters. And do not operate without a filter.
2. Change the filter every 30 days or when the filter turns gray (check the condition of the filter at least once a week).
3. Changing procedure:
 - a) remove the filter cover
 - b) replace the used filter with a new one.
 - c) replace the filter cover.

REPLACEMENT PARTS & OPTIONAL ACCESSORIES

Replacement parts are available through your local dealer.

SPECIFICATIONS

Power	AC 230V, 50 Hz
Power Consumption	180 VA
Medication Capacity	10 ml
Particle Size	0,5 to 10 μm
MMAD	3 μm
Sound Level	Around 55 dBA
Average Nebulization Rate	≥ 0.3 ml/min
Compressor Pressure Range	29 to 52 Psi (200 to 360 KPa / 2 to 3.6 bar)
Operating Pressure Range	11.6 to 19 Psi (80 to 130KPa / 0,8 to 1.3 bar)
Liter Flow Range	4 ~ 7 lpm
Operating Temperature Range	50°F to 104°F (10°C to 40°C)
Operating Humidity Range	10 to 95% RH
Storage Temperature Range	-13°F to 158°F (-25°C to 70°C)
Storage Humidity Range	10 to 95 % RH
Dimension (L x W x H)	172 x 136 x 184 mm
Weigh	1,4 kg
Standard Accessories	Air Tube, Mouthpiece, Filters (5pcs), Adult mask, Child Mask, Nebulizer Bottle, Nosepiece

Protection against electric shock:

A device with protection class II

Protection against SPECIFICATIONS

harmful ingress of water: IPX0.

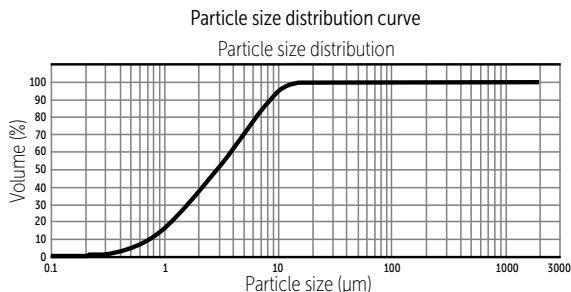
Degree of safety in the presence

of flammable anesthetics or oxygen:

– no AP/APG (not suitable for use

in the presence of flammable anesthetics

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For more information about our products,
please call or email us or visit
www.sanity.pl

INFORMATION CLAUSE

[Information clause for a person who is executing the manufacturer`s warranty]

[Personal Data Administrator]

The Administrator of your personal data is ALBERT POLSKA sp. z o.o. with its registered seat in Dobczyce (Poland), address: ul. Obwodowa 4A, 32-410 Dobczyce, entered into the Register of Entrepreneurs of the National Court Register kept by the District Court for Kraków-Śródmieście in Kraków, 12th Commercial Division of the National Court Register under the number: 0000098464, tax id number: 6811004325, statistical number: 350815320 (hereinafter referred to as the PDA).

[Purpose of processing]

Your personal data will be processed in order for the PDA to fulfill its obligations arising from the warranty granted to you for the purchased product. In addition, your personal data will be processed in order for the PDA to perform its obligations under the tax law and accounting rules. Your personal data will be also processed for legitimate purposes carried out by PDA, i.e.: 1) for the efficient implementation of current PDA activities, including the implementation of current business and administrative processes related to the subject of PDA activities; 2) for the possible establishment, pursuing or defense of claims due to the PDA in relation to the warranty granted to you.

[Legal basis]

The legal basis for the processing of personal data is contained in art. 6 par. 1 letters b), c) and f) of the GDPR. Providing data is voluntary, however it is necessary for the execution of your warranty rights.

[Recipients]

The recipients of your personal data are suppliers of goods and services necessary for the implementation of the abovementioned purposes and entities to which the PDA entrusted the processing of your personal data.

[Data storage period]

Your personal data will be stored until the expiry of warranty and to the extent required by law (i.e. tax law) or to secure any claims. After this time your data will be deleted.

[Rights]

Depending on legal basis of processing of personal data as mentioned hereinbefore, you have the right to access your personal data and the right to rectify them, as well as the right to remove them or limit its processing and the right to object to the processing or to transfer your personal data and the right to object to their processing. You also have the right to lodge a complaint with the supervisory body - the President of the Office for Personal Data Protection.

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